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Please keep these instructions for use so you can refer to them later.

Indications for Use: VEOS moisturizing eye drops refresh and moisturize the eye, also improving comfort of use when wearing contact lenses.

Content: 10 ml eye drops, one instructions for use.

Directions for use:

- Wash hands thoroughly before use.
- Check tamper evident ring is intact before first use.
- Twist and remove the cap from the dropper bottle.
- Instill 2-3 drops into each eye as needed.
- Blink several times after application.
- Replace the cap tightly after use.

In the case of discomfort, irritation, pain, redness or blurred vision, remove contact lenses if you are wearing them, and consult your optician / ophthalmologist **immediately**.

Composition: Sterile isotonic solution containing Sodium Hyaluronate (0.2% w/v), Polyhexanide, Sodium Chloride and Phosphate buffer.

Warnings: ■ Do not use if you are allergic to any ingredient in the product ■ Do not use if you are using any other ophthalmic medications, unless advised by your doctor ■ To avoid contamination of the product, do not touch the tip of the dropper with the eye or any other surface ■ Product is for single patient use ■ Do not use if the safety ring is broken or missing ■ Do not re-sterilize ■ Do not use if product is damaged ■ For external use only ■ Do not mix with other liquids to apply the product ■ If you wear contact lenses, proceed with normal care ■ Keep out of reach and sight of children

Shelf Life and Storage Information: ■ Store between 5 °C to 30 °C and away from direct sunlight ■ Can be used for 90 days after opening ■ Do not use after the expiry date

Disposal: Pour unused solution down a drain. Dispose of empty bottle in accordance with local waste disposal regulations.

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Stericon Pharma Pvt. Ltd.